INSTRUCTIONS FOR USE

eZinject High Flow Catheter

Product # TMI1162-HF

THIS PRODUCT DOES NOT CONTAIN LATEX.

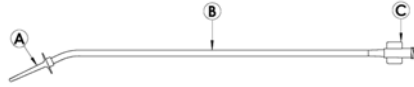
Figure 1

eZinject Catheter™

- A. Cervical Stop
- B. Shaft
- C. Luer

DESCRIPTION:

The **eZinject Catheter High Flow** is a uterine injector for single use. This is a sterile disposable device made of polyvinyl chloride, which meets USP recommendations for implant testing. The product is designed with a single lumen for injection of fluid through a distal tip. The product is slightly curved to facilitate easy entrance to the cervical canal. The distal tip is a tapered design with a cervical stop, preventing excessive penetration. The proximal end is equipped with a luer fitting on a flexible sleeve to accommodate a syringe.

INDICATIONS:

The **eZinject Catheter High Flow** is indicated for such procedures as Sonohysterography. It is designed for "in office use." This device can be applied without cervical dilation or anesthesia if the uterus can be sounded easily with a standard uterine sound. The **eZinject Catheter High Flow** provides for efficient sealing of the cervix when injections of liquid are required.

CONTRAINDICATIONS:

- Pregnancy
- Uterine or tubal infection

WARNINGS:

- To avoid uterine trauma, insert the **eZinject Catheter High Flow** along the natural axis of the cervical canal.

PRECAUTIONS:

- Lubricate distal end for easy insertion.
- Observe manufacturer's directions and precautions pertaining to any liquid media or gas that is being injected into the patient.

ADVERSE REACTIONS:

- Cramping
- Infection

INSTRUCTIONS FOR USE:

1. Check to see that the sterile pouch containing the **eZinject Catheter High Flow** has not been punctured or damaged thus compromising sterility.
2. Using a speculum, expose the cervix and grasp the anterior lip with a single tooth tenaculum.
3. Lubricate the distal tip for easy insertion. Carefully guide the instrument along the natural axis of the cervix to avoid injury.
4. Injected media may be applied with a standard syringe or extenders as needed.
5. Carefully remove the **eZinject Catheter High Flow** after all injecting is completed.

PRODUCT INFORMATION:

Product Number: TMI1162-HF
OD Size: 2mm to 4mm taper
Length: 29cm
Packaging: 10 sterile units per box

MANUFACTURED BY:

CRI
dba Thomas Medical, Inc.
6102 Victory Way
Indianapolis, IN 46278
Telephone: (800) 556-0349
(317) 872-0074
Facsimile: (317) 872-0169

AUTHORIZED REPRESENTATIVE:

CRI Medical Devices Limited
Wexford Enterprise Centre
Strandfield Business Park
Kerlogue, Rosslare Road
Wexford, Ireland

Comments regarding this device can be directed to Attn: Quality Assurance Dept., CRI, 6102 Victory Way, Indianapolis, IN 46278 USA, Fax: (317) 872-0169

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